

Leneva™ TECHNIQUE GUIDE

These general guidelines are designed to serve only as recommendations. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care. Leneva™ should not be applied until excessive exudate or bleeding, acute swelling, and infection are controlled.

PREPARATION OF THE 1.5cc GRAFT SIZE

IN THE BOX - 1.5cc Graft Size:

- Full syringe containing Leneva
- One empty 3cc syringe
- Luer Winged Adapter

1. Prepare injection site.

Debride wound as necessary followed by irrigation.

2. Remove Leneva and Accessory Kit from packaging.

Remove the inner pouch and pass into the sterile field or procedure area.

NOTE:

The surfaces of the outer packaging are NOT sterile.

3. Remove the empty 3cc syringe from the Accessory Kit and fully depress.

4. Remove the Luer Winged Adapter from the Accessory Kit and attach to the empty 3cc syringe.

5. Remove the full syringe containing Leneva from its foil pouch and remove the end cap.



Discard any tissue which may be protruding from the syringe.

6. Attach the syringe containing Leneva to the Luer Winged Adapter on the empty 3cc syringe.

7. Beginning with small increments, pump Leneva back and forth between the two syringes to complete at least 10 full passes.

8. Remove the empty syringe and Luer Winged Adapter.

9. Attach a 20 G or larger bore needle/cannula to the syringe containing Leneva.

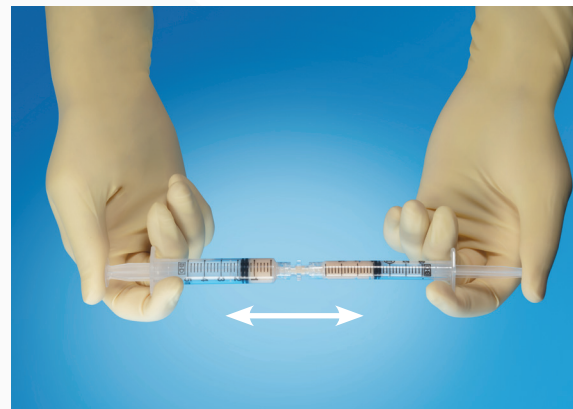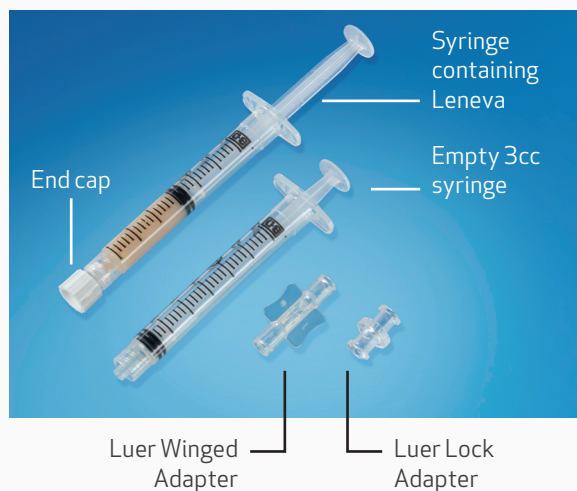

3cc preparation on back →

PREPARATION OF THE 3cc GRAFT SIZE

IN THE BOX - 3cc Graft Size:

- Full syringe containing Leneva
- Three empty 3cc syringes
- Luer Winged Adapter
- Luer Lock Adapter

1. Prepare injection site.

Debride wound as necessary followed by irrigation.

2. Remove Leneva from outer packaging.

Remove the inner pouch and pass the inner pouch into the sterile field or procedure area.

NOTE:

The surfaces of the outer packaging are NOT sterile.

3. Remove the empty 3cc syringes, Luer Lock Adapter, and Luer Winged Adapter from the Accessory Kit.

4. Attach the Luer Lock Adapter to an empty 3cc syringe.

5. Remove the full syringe containing Leneva from its foil pouch and remove the end cap.

Discard any tissue which may be protruding from the syringe. Attach to Luer Lock Adapter on empty syringe

6. Transfer 1.5cc of Leneva into the empty 3cc syringe.

7. Disconnect the 3cc syringe containing Leneva and affix an empty 3cc syringe using the Luer Lock Adapter.

8. Transfer the remaining 1.5cc of Leneva into the empty 3cc syringe.

9. Remove the Luer Lock Adapter and empty 5cc syringe.

10. Attach a Luer Winged Adapter to the third empty 3cc syringe.

11. Prepare the syringes containing Leneva for use following steps 6-8 of the 1.5cc Graft Size Preparation.

